ADCA

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ANVISA nº: 80116050002

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VER.02

Intracranial Clip
For cerebral aneurysm
Titanium
ADCA

Instructions

The logo for ADCA features the letters 'ADCA' in a large, bold, serif font. A thin, grey, curved line arches over the top of the letters, starting from the right side of the 'A' and ending above the 'A'.

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INTRACRANIAL CLIP FOR CEREBRAL
ANEURYSM – TITANIUM ADCA

MEDICAL PRODUCT OF SINGLE USE

Sterile product

Reprocessing strictly forbidden

INDICATION

ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM is only indicated for treatment of cerebral aneurysms, for patients of all ages, being responsibility of the neurosurgeon the assessment of the surgical risk, as well as the choice of the right models to be used for the several types of aneurysms. All information and post-operative care and procedures must follow the instructions given by the neurosurgeons.

RAW MATERIAL

ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM is manufactured with a shaped, tested and approved alloy for surgery implants, according to Standard ISO 5832-3, paramagnetic, stainless and sterilizable, with composition and properties as shown below:

CHEMICAL ANALYSIS OF THE MATERIAL

Element	Composition Limits % (m/m)
Aluminium	5,5 a 6,75
Vanadium	3,5 a 4,5
Iron	0,3 maximum
Oxygen	0,2 maximum
Carbon	0,08 maximum

Nitrogen	0,05 maximum
Hidrogen	0,015 maximum
Titanium	Balance

MECHANICAL PROPERTIES

Alloy form	Bar
Tensile Strength Rm Minimum MPa	860
Conventional Elastic Limit Rp 0,2 Minimum MPa	780
Percentage Elongation after Fracture * A minimum %	10

- * Calibration Length = $5,65 \sqrt{so}$ or 50 mm, where is the original área of the cross-section in square milimeters.
- * Maximum diameter or thickness = 75 mm.

OPENING MOVES TYPES

ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM has the following moves types:

ALLIGATOR: for Standard and Mini types;

ENCIRCLING: for Fenestrated types.

PRESSURE TABLE

TIPE	PRESSURE (g/l)
Mini Permanent	150 to 200
Standard Permanent	200 to 300
Standard Temporary	100 to 150
Mini Temporary	80 to 120
Fenestrated	200 to 300

TYPES OF FRIEZES

The friezes of ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM are of the parallel serrated type, which fit across offering great grip and fully occupy the area involved, thus providing a safe clipping.

DEGRADATION OF CLOSING FORCE

All the production batches of ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM are submitted to the pressure degradation test, which consists of setting in motion a random clip ten consecutive times and then check their pressure loss. None of the released batches has clips with higher than 5% degradation of the actual closing force.

CLEANING AND DESINFECTION

ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM goes through a rigorous cleaning process that reduces its microbial load, of pyrogens and endotoxins, ensuring greater safety in its use.

PACKAGING

The blister, containing the ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM, should be opened inside the operating room, by professionals only, during the surgery as per following procedure:

Open the package, holding the blister so that its label faces down, allowing the visualization of TYVEC's opening area. Once having identified the open-

ing area, open the package firmly pulling the TYVEC with your fingers. Use surgical forceps previously sterilized to remove the clip from its cradle.

Any clip removed from the package and not used, must be discarded, being expressly prohibited to be reused.

STERILIZATION PROCESS

ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM is submitted to sterilization by Ethylene Oxide (ETO), and its packaging ensures a shelf life of five years. Sterilization is carried out in concentration ranging from 550 to 700mg / l and the standard exposure time is 240 minutes at 55° C, with a variation of + or - 5° C. All this service is carried out by a specialized

company and authorized by the competent body.

In order to guarantee the quality of sterilization, ETO residue tests are performed, as well as the sterility test, periodically carried out by specialized and accredited laboratories, proving the efficiency, thus providing complete security.

STORAGE AND CONSERVATION

ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM does not require any special storage care. However, it should not be exposed to extreme heat or cold temperatures, and should be protected from light and moisture. It must be kept in secure locations and protected from the presence of insects and rodents.

TRACEABILITY

ADCA provides together with the packaging of the product, five labels with the trace information of the clips that can be used for fixation on the patient records, identifying the use of the implant by the hospital. Trace tags can also be used in administrative processes of health services or in any other situation that the identification of the product should be required.

DIRECTIONS FOR USE

ADCA'S TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM must be implanted by neurosurgeons who have knowledge of the specific techniques of cerebral aneurysm surgeries. It also must be applied by using specific Applying Forceps for each type of clip.

WARNING

Improper use of applicators can cause serious damages to the clip, leading to poorly functioning and resulting in loss of pressure. The clips should also not be open with fingers or other type of Applying Forceps, as this may lead to pressure change.

ATTENTION: Should the Applying Forceps are damaged, does not use them.

The clip handled, implying in its opening, either by hand or with the applicator must be separated for exchange, as it may present pressure change.

CONTRAINDICATIONS

There is no evidence of any contraindications or restrictions, in compliance with the information in these operating instructions.

NUCLEAR MAGNETIC RESONANCE EXAMINATION

The displacement force and induced torque by MRI environment in the clip were determined according to ASTM F2025-06 and F2213-06 standards, respectively. The results indicated that the induction forces in the titanium clip is not sufficient to move it, once same forces did not overcome the gravitational field values. Therefore, the use of ADCA titanium clips do not cause injury to patients due to the movement of the implant, being its use safe use for environment whose magnetic field does not exceed 3 tesla.

The heating of the clip by radiofrequency induction was evaluated in 1.5 and 3.0 tesla fields as per ASTM F2182-11a. The maximum heating values, taking into account the worst case were

1.1°C and 1.5°C for 1.5 and 3.0 tesla fields, respectively. Transitional temperature rises from 1 to 2°C, in the regions of the implants in patients, while undergoing to magnetic resonance imaging, are considered small ones and safe for magnetic fields studied.

The clips were tested to determine its interference in the images obtained in MRI environments as per ASTM F2119-07. The results showed, taking into account the worst case (MXT-111 - 35,4mm x 7,58mm), distortion as shown below:

Artifact (worst case)	Spin Echo	Gradiente Echo
Length (35,4mm)	7,00mm	8,48mm
Width (7,58mm)	7,49mm	6,97mm

MRI image artifacts affect the object around in accordance with above magnitudes.

Equipment used in the tests: Verio, Siemens Medical Solutions, Erlangen, Germany, 3 Tesla, horizontal static magnetic field, actively shielded.

RE-STERILIZATION

It is fully forbidden the re-sterilization of the clips and all overdue material, as well as with violated packaging must be segregated as unusable material.

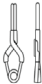
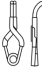
ADVERSE EVENTS

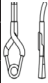
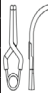
The experience of many years has shown that the aneurysm clips manufactured in titanium do not show safety problems or side effects, provided that the titanium aneurysm clips are handled in accordance with its intended purpose by the surgeon and treated as per its instructions.

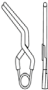
DESCRIPTION OF THE TYPES

The TITANIUM INTRACRANIAL CLIP FOR CEREBRAL ANEURYSM are available in the following types:

Mini Permanent, Standard Permanent, Fenestrated Permanent, Mini Temporary and Standard Temporary.

Type: Mini Clip Permanent				
Ref.	Type	Tip Length	Opening	Image
MXT-00	Straight Thin Tip	3 mm	2,5 mm	
MXT-01		5 mm	4,5 mm	
MXT-04	Curved Thin Tip	5 mm	4,5 mm	
MXT-21		4 mm	3,5 mm	

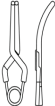
Type: Mini Clip Permanent				
Ref.	Type	Tip Length	Opening	Image
MXT-02	Mid Curved Thin Tip	3 mm	2,5 mm	
MXT-03		5 mm	4,5 mm	
MXT-07		4 mm	3,5 mm	
MXT-05	Straight	3 mm	2,5 mm	
MXT-06		4 mm	3,5 mm	
MXT-08		5 mm	4,5 mm	
MXT-10		6 mm	5,5 mm	
MXT-12		7 mm	6,5 mm	
MXT-11	Curved	6 mm	5,5 mm	
MXT-14		7 mm	6 mm	
MXT-19		3 mm	2,5 mm	
MXT-31		5 mm	4,5 mm	

Type: Mini Clip Permanent				
Ref.	Type	Tip Length	Opening	Image
MXT-09	Mid Curved	5 mm	4,5 mm	
MXT-13		7 mm	6,5 mm	
MXT-18		3 mm	2,5 mm	
MXT-15	Elbow Shaped	5 mm	4,5 mm	
MXT-23		3 mm	3,5 mm	
MXT-16	Bayonet	7 mm	7 mm	
MXT-33		5 mm	5 mm	
MXT-17	45°	7 mm	6,5 mm	

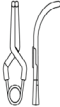
Type: Clip Standard Permanent

Ref.	Type	Tip Length	Opening	Image
MXT-20	Straight	5 mm	4,5 mm	
MXT-30		7 mm	5,5 mm	
MXT-40		9 mm	6,5 mm	
MXT-41		3 mm	3,5 mm	
MXT-43		8 mm	6 mm	
MXT-50		11 mm	7,5 mm	
MXT-51		10 mm	7 mm	
MXT-53		12 mm	8 mm	
MXT-60		13 mm	8,5 mm	
MXT-70		15 mm	9,5 mm	
MXT-80		20 mm	11 mm	
MXT-84		23 mm	12 mm	
MXT-85		25 mm	13 mm	

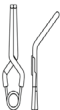
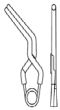
Type: Clip Standard Permanent

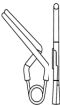
Ref.	Type	Tip Length	Opening	Image
MXT-22	Mid Curved	5 mm	4,5 mm	
MXT-32		7 mm	5,5 mm	
MXT-42		9 mm	6,5 mm	
MXT-52		11 mm	7,5 mm	
MXT-62		13 mm	8,5 mm	
MXT-63		3 mm	3,5 mm	
MXT-65		8 mm	6 mm	
MXT-66		10 mm	7 mm	
MXT-67		12 mm	8 mm	
MXT-72		15 mm	9,5 mm	

Type: Clip Standard Permanent

Ref.	Type	Tip Length	Opening	Image
MXT-24	Curved	5 mm	4,5 mm	
MXT-34		7 mm	5,5 mm	
MXT-44		9 mm	6,5 mm	
MXT-54		11 mm	7,5 mm	
MXT-64		13 mm	8,5mm	
MXT-68		3 mm	3,5 mm	
MXT-69		10 mm	7 mm	
MXT-71		12 mm	8 mm	
MXT-74		15 mm	9,5 mm	

Type: Clip Standard Permanent

Ref.	Type	Tip Length	Opening	Image
MXT-25	45°	5 mm	5 mm	
MXT-35		7 mm	5,5 mm	
MXT-45		9 mm	6,5 mm	
MXT-55		11 mm	7,5 mm	
MXT-83		10 mm	7 mm	
MXT-26	Elbow Shaped	5 mm	5 mm	
MXT-36		7 mm	5,5 mm	
MXT-46		9 mm	6,5 mm	
MXT-56		11 mm	8 mm	
MXT-73		3 mm	3,5 mm	
MXT-75		8 mm	6 mm	
MXT-76		10 mm	7 mm	
MXT-77		12 mm	8 mm	

Type: Clip Standard Permanent				
Ref.	Type	Tip Length	Opening	Image
MXT-27	Angled	5 mm	4,5 mm	
MXT-37		7 mm	5,5 mm	
MXT-47		9 mm	6,5 mm	
MXT-57		11 mm	7,5 mm	
MXT-28	Bayonet	5 mm	5,5 mm	
MXT-38		7 mm	6,5 mm	
MXT-48		9 mm	7 mm	
MXT-58		11 mm	8 mm	
MXT-78		8 mm	6 mm	
MXT-79		10 mm	7 mm	

Type: Clip Standard Permanent

Ref.	Type	Tip Length	Opening	Image
MXT-29	90°	5 mm	4,5 mm	
MXT-39		7 mm	5,5 mm	
MXT-49		9 mm	6,5 mm	
MXT-59		11 mm	7,5 mm	
MXT-81		8 mm	6 mm	
MXT-82		10 mm	7 mm	

Type: Clip Fenestrated Permanent

Ref.	Type	Tip Length	Opening	Ring	Image
MXT-86	Straight	3 mm	4 mm	3,5 mm	
MXT-87		8 mm	5 mm		
MXT-88		10 mm	6,5 mm		
MXT-100		6 mm	5 mm		
MXT-101		9 mm	5,5 mm		
MXT-102		12 mm	6,5 mm		
MXT-103		14 mm	7,5 mm		
MXT-104		16 mm	8 mm		
MXT-105		18 mm	8,5 mm		

Type: Clip Fenestrated Permanent

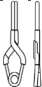
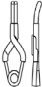
Ref.	Type	Tip Length	Opening	Ring	Image
MXT-92	Straight	3 mm	4 mm	5 mm	
MXT-93		8 mm	6 mm		
MXT-94		10 mm	7 mm		
MXT-106		6 mm	5 mm		
MXT-107		9 mm	6 mm		
MXT-108		12 mm	7 mm		
MXT-109		14 mm	8 mm		
MXT-110		16 mm	8,5 mm		
MXT-111		18 mm	9 mm		

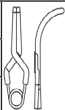
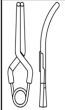
Type: Clip Fenestrated Permanent					
Ref.	Type	Tip Length	Opening	Ring	Image
MXT-89	45°	8 mm	5 mm	3,5 mm	
MXT-112		5 mm	4,5 mm		
MXT-113		7 mm	5 mm		
MXT-114		10 mm	6 mm		
MXT-115		12 mm	7 mm		
MXT-95	45°	3 mm	4 mm	5 mm	
MXT-96		8 mm	5,5 mm		
MXT-116		5 mm	5 mm		
MXT-117		7 mm	5,5 mm		
MXT-118		10 mm	6,5 mm		
MXT-119		12 mm	7,5 mm		

Type: Clip Fenestrated Permanent

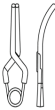
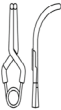
Ref.	Type	Tip Length	Opening	Ring	Image
MXT-91	90°	8 mm	5 mm	3,5 mm	
MXT-120		5 mm	4,5 mm		
MXT-121		7 mm	5 mm		
MXT-122		10 mm	6 mm		
MXT-123		12 mm	7 mm		

Type: Clip Fenestrated Permanent					
Ref.	Type	Tip Length	Opening	Ring	Image
MXT-98	90°	3 mm	4 mm	5 mm	
MXT-99		8 mm	5,5 mm		
MXT-124		5 mm	5 mm		
MXT-125		7 mm	5,5 mm		
MXT-126		10 mm	6,5 mm		
MXT-127		12 mm	7,5 mm		

Type: Mini Clip Temporary				
Ref.	Type	Tip Length	Opening	Image
MXT-130	Straight Thin Tip	3 mm	2,5 mm	
MXT-131		5 mm	4,5 mm	
MXT-132	Mid Curved Thin Tip	5 mm	4,5 mm	
MXT-134	Curved Thin Tip	5 mm	4,5 mm	
MXT-133	Straight	5 mm	4,5 mm	
MXT-135		7 mm	6,5 mm	
MXT-138		9 mm	7 mm	

Type: Mini Clip Temporary				
Ref.	Type	Tip Length	Opening	Image
MXT-129	Curved	5 mm	4,5 mm	
MXT-137		7 mm	6,5 mm	
MXT-139		9 mm	7 mm	
MXT-136	Mid Curved	7 mm	6,5 mm	

Type: Clip Standard Temporary				
Ref.	Type	Tip Length	Opening	Image
MXT-140	Straight	7 mm	5,5 mm	
MXT-143		9 mm	6,5 mm	
MXT-146		15 mm	9,5 mm	
MXT-149		20 mm	11 mm	
MXT-152		5 mm	4,5 mm	
MXT-153		8 mm	6 mm	
MXT-154		10 mm	7 mm	
MXT-155		12 mm	8 mm	

Type: Clip Standard Temporary				
Ref.	Type	Tip Length	Opening	Image
MXT-141	Mid Curved	7 mm	5,5 mm	
MXT-144		9 mm	6,5 mm	
MXT-147		15 mm	9,5 mm	
MXT-150		20 mm	11 mm	
MXT-156		5 mm	4,5 mm	
MXT-157		8 mm	6 mm	
MXT-158		10 mm	7 mm	
MXT-142	Curved	7 mm	5,5 mm	
MXT-145		9 mm	6,5 mm	
MXT-148		15 mm	9,5 mm	
MXT-151		20 mm	10 mm	
MXT-159		5 mm	4,5 mm	
MXT-160		10 mm	7 mm	

Type: Clip Standard Temporary				
Ref.	Type	Tip Length	Opening	Image
MXT-161	Elbow Shaped	5 mm	5 mm	
MXT-162		8 mm	6,5 mm	
MXT-163		10 mm	7,5 mm	

PRESENTATION

ADCA Clip is sterilized, individually wrapped in plastic blister, duly sealed with Tyvec®. Each pack contains 1 clip for aneurysm. The sterile packaging is wrapped in a paper cartridge with an Acetat display, which protects the primary packaging and allows visualization of the implant.

TRANSPORTATION

For the transportation, the paper cartridges are packaged in tough cardboard boxes.

SHELF LIFE (VALIDITY)

The ADCA clips are valid for 5 years, as of its sterilizing date. After such date, the clips are considered improper for use and must be discarded.

EXPLANATION ABOUT PRESENTED GRAPHICS, SYMBOLS AND ABBREVIATIONS

	Re-usage strictly forbidden
	Do not sterilize again
	Warnings and Precautions
	Sterile by Ethylene Oxide
	Product Lot Code
	Product Serial Number
	Date of manufacturing

	To be used until (Validity)
	Catalog Reference Number
	See directions before use
	Do not use if packaging is damaged
	Keep away from Sun Light and Heating
	Keep away from moisture
	Product Manufacturer

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