ADCA

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ANVISA nº: 80116050003

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VER. 02

**Titanium Intracranial
Reinforcing Clip
(Booster Clip)
For cerebral aneurysm**

Instructions

The logo for ADCA features the letters 'ADCA' in a large, bold, serif font. A thin, light gray curved line arches over the top of the letters, starting above the 'A' and ending above the 'A'.

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STANDARD PERMANENT REINFORCING CLIP – TITANIUM

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM

MEDICAL PRODUCT OF SINGLE USE

Sterile product

Reprocessing strictly forbidden

INDICATION

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM is only indicated for treatment of cerebral aneurysms, for patients of all ages, being responsibility of the neurosurgeon the assessment of the surgical risk, as well as the choice of the right models to be used for the several types of aneurysms.

All information and postoperative care and procedures must follow the instructions given by the neurosurgeons.

RAW MATERIAL

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM is manufactured with a shaped, tested and approved alloy for surgery implants, according to Standard ISO 5832-3, paramagnetic, stainless and sterilizable, with composition and properties as shown below:

CHEMICAL ANALYSIS OF THE MATERIAL

Element	Compositions Limits % (m/m)
Aluminium	5,5 a 6,75
Vanadium	3,5 a 4,5
Iron	0,3 maximum

Element	Compositions Limits % (m/m)
Oxygen	0,2 maximum
Carbon	0,08 maximum
Nitrogen	0,05 maximum
Hydrogen	0,015 maximum
Titanium	Balance

MECHANICAL PROPERTIES

Alloy form	Bar
Tensile Strength Rm Minimum MPa	860
Conventional Elastic Limit Rp 0,2 Minimum MPa	780
Percentage Elongation after Fracture * A minimum %	10

* Calibration Length = $5,65 \sqrt{S_0}$ or 50 mm, where is the original área of the cross-section in square milimeters.

* Maximum diameter or thickness = 75 mm.

OPENING MOVE TYPE

THE TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM has the following move type:

ALLIGATOR: for Standard type;

PRESSURE AND OPENING TABLE

TYPE	SIZE (mm)	PRESSURE (g/f)	OPENING (mm)
Standard Permanent	3	200 to 230	5

TYPES OF FRIEZES

THE TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM does not have friezes, but fitting orifice only.

DEGRADATION OF CLOSING FORCE

All the production batches of ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM are submitted to the pressure degradation test, which consists of setting in motion a random clip ten consecutive times and then check their pressure loss. None of the released batches has clips with higher than 5% degradation of the actual closing force.

CLEANING AND DESINFECTION

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM goes through a rigorous cleaning process that reduces its microbial load, of pyrogens and endotoxins, ensuring greater safety in its use.

PACKAGING

The blister, containing the ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM, should be opened inside the operating room, by professionals only, during the surgery as per following procedure:

Open the package, holding the blister so that its label faces down, allowing the visualization of TYVEC's opening area. Once having identified the opening area, open the package firmly pulling the TYVEC with your fingers. Use surgical forceps previously sterilized to remove the clip from its cradle.

Any clip removed from the package and not used, must be discarded, being expressly prohibited to be reused.

STERILIZATION PROCESS

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM is submitted to sterilization by Ethylene Oxide (ETO), and its packaging ensures a shelf life of five years. Sterilization is carried out in concentration ranging from 550 to 700mg/l and the standard exposure time is 240 minutes at 55° C, with a variation of + or - 5° C. All this service is carried out by a specialized company and authorized by the competent body.

In order to guarantee the quality of sterilization, ETO residue tests are performed, as well as the sterility test, periodically carried out by specialized and accredited laboratories, proving the efficiency, thus providing complete security.

STORAGE AND CONSERVATION

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM does not require any special storage care. However, it should not be exposed to extreme heat or cold temperatures, and should be protected from light and moisture. It must be kept in secure locations and protected from the presence of insects and rodents.

TRACEABILITY

ADCA provides together with the packaging of the product, labels with the trace information of the clips that can be used for fixation on the patient records, identifying the use of the implant by the hospital. Trace tags can

also be used in administrative processes of health services or in any other situation that the identification of the product should be required.

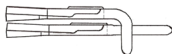
DIRECTIONS FOR USE

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM must be implanted by neurosurgeons who have knowledge of the specific techniques of cerebral aneurysm surgeries. It also must be applied by using specific Applying Forceps for each type of clip.

Couple the ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL ANEURYSM on the other

CLIP STANDARD PERMANENT STRAIGHT higher than 15 mm as illustrated below:

Reinforcing Clip

IMPORTANT REMARK: The Reinforcing Clip should never be used alone. It must be coupled to another Clip Standard Straight.

WARNING

ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIP FOR CEREBRAL must only be implanted by neurosurgeons who have knowledge of the specific techniques of cerebral aneu-

rysm surgeries. It also must be applied by using specific Applying Forceps for each type of clip.

Improper use of applicators can cause serious damages to the clip, leading to poorly functioning and resulting in loss of pressure. The clips should also not be open with fingers or other type of Applying Forceps, as this may lead to pressure change.

ATTENTION: Should the Applying Forceps are damaged, does not use them.

The clip handled, implying in its opening, either by hand or with the applicator must be separated for exchange, as it may present pressure change.

CONTRAINDICATIONS

There is no evidence of any contraindications or restrictions, in compliance with the information in these operating instructions.

NUCLEAR MAGNETIC RESONANCE EXAMINATION

The displacement force and induced torque by MRI environment in the clip were determined according to ASTM F2025-06 and F2213-06 standards, respectively. The results indicated that the induction forces in the titanium clip are not sufficient to move it, once same forces did not overcome the gravitational field values. Therefore, the use of ADCA titanium clips do not cause injury to patients due to the movement

of the implant, being its use safe use for environment whose magnetic field does not exceed 3 tesla.

The heating of the clip by radiofrequency induction was evaluated in 1.5 and 3.0 tesla fields as per ASTM F2182-11a. The maximum heating values, taking into account the worst case were 1.1°C and 1.5°C for 1.5 and 3.0 tesla fields, respectively. Transitional temperature rises from 1 to 2°C, in the regions of the implants in patients, while undergoing to magnetic resonance imaging, are considered small ones and safe for magnetic fields studied.

The clips were also tested to determine its interference in the images obtained in MRI environments as per ASTM F2119-07. The results showed, taking

into account the worst case (MXT-111 - 35,4mm x 7,58mm), distortion as shown below:

Artifact (worst case)	Spin Echo	Gradiente Echo
Length (35,4mm)	7,00mm	8,48mm
Width	7,49mm	6,97mm

MRI image artifacts affect the object around in accordance with above magnitudes.

Equipment used in the tests: Verio, Siemens Medical Solutions, Erlangen, Germany, 3 Tesla, horizontal static magnetic field, actively shielded.

RE-STERILIZATION

It is fully forbidden the re-sterilization of the clips and all overdue material, as well as with violated packaging must be segregated as unusable material.

ADVERSE EVENTS

The experience of many years has shown that the aneurysm clips manufactured in titanium do not show safety problems or side effects, provided that the titanium aneurysm clips are handled in accordance with its intended purpose by the surgeon and treated as per its instructions.

DESCRIPTION OF THE TYPES

The ADCA'S TITANIUM INTRACRANIAL REINFORCING CLIPS FOR CEREBRAL have just one type and size, as per the following table:

CODE	PICTURE	DESCRIPTION
MXT-128		90°, 3 mm

PRESENTATION

ADCA Clip is sterilized, individually wrapped in plastic blister, duly sealed with Tyvec®. Each pack contains 1 clip for aneurysm. The sterile packaging is wrapped in a paper cartridge with an Acetat display, which protects the primary packaging and allows visualization of the implant.

TRANSPORTATION

For the transportation, the paper cartridges are packaged in tough cardboard boxes.

SHELF LIFE (VALIDITY)

The ADCA clips are valid for 5 years, as of its sterilizing date. After such date, the clips are considered improper for use and must be discarded.

EXPLANATION ABOUT PRESENTED GRAPHICS, SYMBOLS AND ABBREVIATIONS

	Re-usage strictly forbidden
	Do not sterilize again
	Warnings and Precautions
	Sterile by Ethylene Oxide
	Product Lot Code
	Product Serial Number
	Date of manufacturing

	To be used until (Validity)
	Catalog Reference Number
	See directions before use
	Do not use if packaging is damaged
	Keep away from Sun Light and Heating
	Keep away from moisture
	Product Manufacturer

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